# LuxaTray

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## English

### Product Description

LuxaTray is a reusable light-shielding tray for the mixing and the temporary storage of light- and contamination-sensitive materials (e.g. composites, solutions, ceramic materials, veneers). LuxaTray protects up to 5 materials from premature polymerization.

### Basic Safety Instructions

- For dental use only!
- Keep out of reach of children!

### Instructions for Use

- ▶ To avoid contact of the material with the cover, the material quantity per well should not exceed a fill level of 10 mm.
- ▶ After treatment, remove all of the materials from the LuxaTray and clean or replace the LuxaTray insert.
- ▶ Observe the manufacturer's instructions of the materials used.
- ▶ Avoid direct exposure of the LuxaTray to light. Always keep the filled LuxaTray closed and close it immediately after removing any materials in order to avoid premature polymerization of the materials in the tray.

### Recommended Use

#### Prepare LuxaTray

- ▶ Align the two openings at the underside of the LuxaTray insert with the guide-ways of the LuxaTray and locate the LuxaTray insert into position by applying light pressure (Fig. 1).

#### Fill/remove materials

1. Load the wells with the materials needed.

**Please note:** For a safe allocation of the materials, the wells of the LuxaTray insert are numbered from 1 to 5.

2. Remove the desired amount of material as needed.
3. Close the cover.

### Cleaning and Disinfection

- ▶ After treatment, remove all of the materials using a conventional gauze swab or a tissue and, if necessary, a solvent.
- ▶ Disinfection can be carried out using a commercially available dipping or wiping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute).
- ▶ The LuxaTray inserts are reusable after disinfection, although they should be replaced after 5–10 uses or if visible signs of wear, e.g. scratches, appear.

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