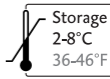
# Constic

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## English

### Product description

Constic is a self-etching, self-adhesive, radiopaque, flowable composite which is light-cured. The composite is immediately ready for use because the preparatory steps of etching, priming and bonding are contained in Constic.

### Indications

- Small restorations of class I + III
- Restorations of class V
- Cavity liner for class I + II
- Fissure sealing
- Extended fissure sealing
- Repairs of composite restorations
- Modifications to temporaries and long-term temporaries
- Blocking out and filling of undercuts in cavities
- Small occlusal primary tooth cavities

### Contra-indications

- ▶ Do not use Constic in the case of allergies to any of the components or in the event of contact allergies.
- ▶ Application of the material is contraindicated if dry isolation or the recommended application technique is not possible.
- ▶ Do not apply Constic to the exposed pulp.
- ▶ Do not use Constic on etched dentine.

### Basic Safety Instructions

- For dental use only!
- Keep out of reach of children!
- Avoid contact with the skin! In the event of inadvertent skin contact, wash the affected area thoroughly with soap and water. Wear suitable gloves.
- Avoid contact with the eyes! In the event of inadvertent contact with the eyes, rinse thoroughly with plenty of water immediately and consult a physician if necessary.

### Side effects

Isolated cases of hypersensitivity or contact allergy to components of the material cannot be excluded.

### Interactions with other materials

- ▶ Materials containing eugenol, moisture and oily air may inhibit polymerization at the point of contact. Avoid materials containing eugenol, moisture and air containing oil!
- ▶ The use of cationic mouthwashes, plaque indicators and chlorhexidine may lead to discoloration.

### Instructions for use

- ▶ Prior to application, bring the material to room temperature (15 to 25 °C / 60 to 75 °F).
- ▶ Light-curing units should have an output of 450 nm and should be checked regularly. The light intensity should be a minimum of 400 mW/cm<sup>2</sup>. Observe the manufacturer's instructions for the light unit.
- ▶ Position the curing light as close to the material as possible.
- ▶ Do not remove the oxygen-inhibited layer because this is required for the bond with the next layer.
- ▶ When using Constic in conjunction with other materials, follow the relevant manufacturer's instructions.

### Recommended Use

#### Minor restorations of class I, III, V and underfillings of class I and II

1. Determine tooth color and select the appropriate shade.
2. Prepare the cavity in accordance with the general rules for adhesive procedures. Bevel the enamel edges if necessary. Protect pulp in deep cavities with appropriate preparations.

**Note:** DMG recommends the use of a rubber dam.

3. Clean tooth, remove all cleaning residue with water spray.

**Note:** Cleaning with alcohol may cause overdrying of the dentine.

4. Dry the tooth using water- and oil-free air in order to avoid overdrying of the dentine. A moist layer must remain on the surface of the tooth.
5. Apply the material onto the cavity surface with the aid of the Luer-Lock-Tip by pressing the syringe and massage a thin layer (≈ 0.5 mm) into the entire surface of the cavity wall for 25 s using the brush.
6. Remove any excess, if necessary.
7. Light-cure for 20 s.
8. Layer the material as follows:
  - ▶ Small restorations of class I, III, V: Use the material to build up layers with a maximum 2 mm layer thickness and cure each layer for 20 s.
  - ▶ Underfillings of classes I and II: Use the material to build up 1 layer with a maximum 1 mm layer thickness and light-cure for 20 s.

**Note:** It is not necessary to repeat massaging of the individual layers.

9. Remove excess, contour the filling surface and finish using suitable instruments.
10. Use polishing paste to polish the filling surface and neighboring surfaces. Use a polishing cup or polishing disk for polishing work.

#### Fissure sealing and extended fissure sealing

1. For extended fissure sealing, widen exceedingly narrow and ampullate fissures.
2. Clean enamel with fluoride-free paste, remove all cleaning residue with water spray and dry the tooth with water- and oil-free air. In doing so, leave a moist surface on the enamel.
3. Determine tooth color and select the appropriate shade.

**Note:** DMG recommends enamel etching in addition to cleaning. To do this, etch enamel for 15–30 s with 37% phosphoric acid gel, rinse for 15 s with water and dry with oil- and water-free air. In doing so, leave a moist surface on the enamel.

4. Apply the material onto the surface of the fissure or cavity wall with the aid of the Luer-Lock-Tip by pressing the syringe and massage a thin layer (≈ 0.5 mm) into the entire surface for 25 s using the brush.
5. Remove any excess, if necessary.
6. Light-cure for 20 s.
7. Where necessary, use the material to build up layers with a maximum 2 mm layer thickness and light-cure for 20 s.

**Note:** It is not necessary to repeat massaging of the individual layers.

8. Check occlusion and correct if necessary.

### Disinfection

The brush holder is reusable after disinfection, although it should be replaced if visible signs of wear, e.g. scratches, appear. Disinfection can be carried out using a commercially available wiping or dipping disinfectant. DMG recommends using only disinfecting solutions listed by the RKI (Robert Koch Institute).

### Composition

Barium glass in a Bis-GMA-based matrix from dental resins. Pigments, additives and catalyst. Filler content: 65 wt.-% = 38 vol.-%. The range of variation of the inorganic filler particles is between 0.02 and 2.3 µm.

### Storage and shelf life

- ▶ Store in the refrigerator (2 to 8 °C/ 36 to 46 °F)!
- ▶ Reseal syringes after use!
- ▶ Do not use after the expiration date!

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